SECUTEST (ST) BASE(10) / PRO & SECULIFE ST BASE(25) / PRO

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3-349-807-15

Amendments and Addenda for Firmware Update

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Attention!

Use IZYTRONIQ version 4.7.0.1 or higher!

- SECULIFE ST PRO (orange rotary switch positions) – Bug from firmware 3.3.0 has been fixed: Measurement of patient leakage current/applied part leakage current of first applied part group now works correctly.
- Bug from firmware 3.3.0 has been fixed: Special characters can be used in file names on USB sticks again.
- Change: Settings under “SETUP > Autom. measurement.” have been rearranged.
- New feature “AutoPrint” has been added: Print reports automatically at the of a test or when a test is saved to the database. (Only with SECUTEST DB COMFORT (Z853S or feature KD01).)

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Use IZYTRONIQ version 4.7 or higher!

Attention!

Changes to the standards and the test sequences in the instrument!

The update changes rotary switch position assignments (AUTO, A1, A2...) and how the standards are shown at the instrument, in the test data and in the test reports!

Due to the fact that the standard display can't be changed retroactively, you'll have to reconfigure your instrument immediately after updating the firmware. Read and comply with the information and instructions included in ⇨ "Changes to the Standards and Test Sequences" (see below)!

Changes to the Standards and Test Sequences

Due to changes in the applicable standards, this firmware updates the instrument as follows:

- New test sequences: EN 50678 (VDE 0701), EN 50699 (VDE 0702), EN 62368, EN 62911 (only with feature KE01 or enabling of Z853U.)
- Selection of national standard designations amended and extended
- Optimized test sequence selection

These updates permit adaptation to the test instrument's specific area of application. As a result, it will be necessary to reconfigure the test sequences in your device after the firmware update, because rotary switch position assignments and how the standards are shown will be changed.

Proceed as follows:

1. Switch the test instrument on.

Note!

Steps 2 and 3 (update and enabling of Z853U) are only required for older instruments.

Go directly to step 4 for newly purchased instruments.

2. Install the firmware update to your instrument. Corresponding instructions can be found in the operating instructions for the instrument.
↳ Your device is updated with the new features.
3. If you have purchased the new test sequences (enabling of Z853U), install them now. To do so, enter the code under **Setup 1/3 > System > Features > Enter key manually** and acknowledge with the green checkmark. Refer to the instrument's operating instructions for detailed information on function extensions.
↳ The extension is enabled.

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GB

4. Select **SETUP > Autom. measurement.** and scroll to page **4/7**. The standards and test sequences available for the instrument are displayed.

5. Select either a national designation for each standard or deactivate the standard with **off** if it's not required. (Deactivating unused or obsolete standards increases clarity on the instrument).

Example: **SETUP > Autom. measurement. 4/7 > VDE 0701-0702 > off / OVE E 8701 / SNR 462638 / VDE 0701-0702**

Select each standard via the softkey followed by the desired designation or deactivation with the softkey as well. The menu is returned to the standards display in **Autom. measurement**. Repeat this procedure for all standards. Finally, confirm with the green checkmark.

↳ The instrument is restarted and the settings are applied. This process may take a while.

All rotary switch positions (**AUTO, A1, A2...**) are automatically updated accordingly (assigned to the standards which have not been switched "off"). From now on, the selected national standard designations will also be used for representation of the standards at the instrument, as well as in the test data and test reports. Designations of standards in tests that have already been saved remain unchanged.

6. Check the rotary switch positions. Change them as required:

– Select an orange rotary switch position (**AUTO, A1, A2...**) – the initial page of the respective test sequence is displayed, i.e. the default test sequence.

– Select classification parameters .

The **Classification Parameters 1/2** page appears.

The standard which is currently assigned to the respective rotary switch position is displayed under **Standard / Test Sequence**.

– Select **Standard / Test Sequence**.

The **Standard / Test Sequence** page appears. All standards available on the instrument are displayed here according to the selected setting (off / national designation – see above).

– Select the desired standard.

The menu is returned to the **Classification Parameters 1/2** page.

– Finally, confirm with the green checkmark .

↳ The settings are saved.

– Repeat the above described procedure for each of the respective rotary switch positions.

↳ Your instrument is then ready for service again.

Observe the following information regarding amendments and addenda.

Changes and Added Features

- Selection of 'Measurement sequences' in the setup changed: Individual test standards can now be (de)activated. All rotary switch assignments (Auto/A1/A2...) are automatically adapted accordingly. The function enables adaptation to the specific application area of the test

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GB

instrument. We recommend an adjustment directly after the firmware update.

- Selection of standard designation added: Determines with which designation the standards are displayed in the test sequences integrated in the test instrument (e.g. national designations). The selected standard designation appears in the test report. The setting is not retroactive; standard designations in already saved tests are unchangeable. The setting can be found in the setup under 'Autom. measurement'.
- New test sequences added: EN 50678 (VDE 0701), EN 50699 (VDE 0702), EN 62368, EN 62911. The test standards can be activated in the setup under 'Autom. measurement' (only with option KE01 or activation Z853U). If necessary, deactivate the previous standards for a better overview of the test sequences in the selection list.
- EN 50678 (VDE 0701), EN 50699 (VDE 0702): Limit value formula for protective conductor resistance depending on conductor cross-section added. Input of conductor cross-section and length is only required if 100 mΩ is exceeded (smallest possible value = 100 mΩ). (Only with option KE01 or activation Z853U.)
- All integrated test sequences with suffix '-ExtC': Added new sequence parameter 'Duration RINS'. When set to '0s', this enables comprehensive testing of long extension cables for insulation faults while the cable is being moved.
- All integrated test sequences with suffix '-PRCD': Sequence parameter 'IPE mode' now affects all IPE test steps correctly. (fix)
- All integrated test sequences except for IEC 60601: Test speed improved (default duration for RPE measurement reduced to 3 seconds).
- Leakage current measurements: Safety shutdown changed; the shutdown occurs when the probe current exceeds approx. 30 mA (version 3.2.0 and earlier: 12 mA). The thresholds of the residual current safety shutdown remain unchanged
- VDE 0701-0702(-EDV) / NEN 3140(-EDV) / ÖVE E 8701(-EDV) / SNR 462638(-EDV): Sequence parameter 'IT mode (active)' affects only IT measurements on DUTs without protective earth connector (PC II, PC II+III).
- VDE 0701-0702 / NEN 3140 / ÖVE E 8701 / SNR 462638 / EN 62353 / EN 60974-4 (and if installed IEC 60601): The default current for the RPE test step in integrated test sequences has been changed to ± 200 mA= (was 200mA~).
- VDE 0701-0702 / IEC 60974-4: Cable lengths extended to 100 m (for the formula-based determination of the protective conductor resistance limit).
- IEC 62353: Measurement of insulation resistance on applied parts is off by default (even if rotary switch configuration has been changed).
- IEC 60601 (only with option KA01): Leakage current measurements with special test conditions is off by

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GB

- default (even if rotary switch configuration has been changed).
- IEC 60601 (only with option KA01): Measurement of insulation resistance always off as standard (even if rotary switch is reassigned).
- IEC 60601 (only with option KA01): Default measurement duration for RPE changed to 5 seconds (was 4 seconds).
- SECULIFE ST PRO/green rotary switch positions: Parameter 'Polarity' is not displayed in a separate box any more (actual polarity is still displayed in the softkey icon). Thus the 'AP' parameter is now always displayed in the same spot.
- Test reports (HTML file/thermal printer/AQV): When saving a passed test, the next test due date will be calculated. The new parameter SETUP - 'Autom. measurement' - 'Standard test interval' is used for the calculation (default value but is adjustable). The next test date is also added to the test report, you can disable this feature under SETUP.
- New: Calculation of next test due date individually for each DUT using the database field 'interval'. (Only available with feature Z853R or option KB01.)
- HTML test reports: File naming scheme changed. For single tests/measurement: [ID of DUT]-[sequence name]-[test time]-[offline/online].html (previously: print time). For all tests/measurements of a DUT: [ID of DUT]-Multi-[offline/online].html.
- HTML test reports: Locations in location hierarchy of DUT are now displayed correctly. (fix)
- Faster file operations (database backup/restore, saving HTML test reports).
- Faster device start-up.

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(14/3.21)

Attention!

IZYTRONIQ-Update required.
Use at least version 4.7!

- **PRCDs:** Trip time measurement improved.
- **IA – Leakage Current from the Applied Part:** Determination of the secondary measured values improved.
- **R_{PE} – Measuring Protective Conductor Resistance 10 A/25 A:** within multi measurement steps of automatic test: high test current is only active while the evaluation period is running. (Only available with option G01 or G02.)
- **R_{PE} – Protective conductor resistance measurement with 200 mA AC:** Testing current frequency fixed.
- **User-defined sequences:** Visual inspections can now be executed when the "Meas. at IT-mains" option is active.
- **User-defined sequences:** Several directly successive measurement test steps now work in the "I_{PA} – Patient auxiliary current" measurement. (Only with option J01.)
- **Built-in test sequences:** Default protection class changed from "PC I + II" to "PC I".
- **IEC 60974-4:** I_T measurement at PC II parts can be deactivated individually.
- **Internal memory and external USB memory:** Writing speed has been increased.
- **Test reports:** Error handling of large image files used as "logo" improved.
- **HTML test reports:** New feature to display *all* values of a multi measurement. (Default setting = Display all values.)
- **HTML test reports:** Recalibration date added to report. (For each test saved with this version or later.)
- **Export:** Data with incompletely saved test results can now be exported as .secu-file or to a PC.
- **Thermal printer Z721S:** Recalibration date on test reports fixed.

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- L1 continuity check (VL2E-Adapter):** Result strings have changed — Failed => L1/L2/L3/N faulty.
- Automatic test sequences:** Automatic switch-off of test socket when current consumption is reduced/low can be enabled in SETUP. (Only available with feature "Z853S" or option "KD01").
- Automatic test sequences:** The hint "Please put DUT into operation" (which appears after switching the test socket to mains voltage) can be deactivated (SETUP > Autom. measurement.).
- Automatic test sequences:** The hint "Please switch off DUT on its main switch or push confirm if already off." (which appears before switching the test socket to mains voltage) only will be displayed for DUTs with low internal resistance.
- Automatic test sequences:** If "SETUP > At limit viol. > Continue" is selected and a limit violation occurs during certain RPE or RINS test steps, a warning pops up and must be confirmed. (Only available with feature "Z853S" or option "KD01").
- Automatic test sequences:** Optimization of the automatic test sequences.
- User-defined test sequences:** New check available: "Switchable check" determines whether the internal resistance of the connected DUT is low enough, so that mains power can be switched via the internal relays (i.e. without switching off the DUT power switch).
- Continuity test with EL1-adapter:** Efficiency and safety level has been improved.
- PRCD trip time measurement:** Renamed display name of measuring function and name of measured value — tB => tPRCD and tB => tt.
- PRCD trip time measurement within automatic test sequences:** Entry in comment field of test result for limit violation fixed.
- VDE 0701-0702:** Input of cable length/cross section is no longer required when measured value is < 0.2 Ω.
- VDE 0701-0702/NEN 3140/ÖVE E 8701/SNR 462638:** Cable cross-section of 25 mm² can be entered for RPE limit value calculation (> 16 A).
- VDE 0701-0702-PRCD/NEN 3140-PRCD/ÖVE E 8701-PRCD/SNR 462638-PRCD:** Test sequence for PRCD-K types has been fixed.
- VDE 0701-0702-ExtC/NEN 3140-ExtC/ÖVE E 8701-ExtC/SNR 462638-ExtC with EL1-adapter connection:** Efficiency and safety level of the continuity test has been improved.

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GB

Firmware Update to Version 3.1.0 (continued)

- **NEN 3140:2018:** Now available at a charge.
- **Integrated test sequences for extension cords:** Visual inspection questions reduced.
- **Probe check:** Parameter values for probe selection have changed (affecting remote control and user-defined test sequences).
- **USB devices:** USB-Icons in the header will be displayed with a digit indicating the count of connected devices of this type.
- **Bluetooth:** Bluetooth-settings (pairings, Bluetooth device name, etc.) are lost when updating the firmware from 3.0.0 (or older) to 3.1.0 (or newer). Enter Bluetooth device name again. Re-establish Bluetooth pairings with required devices/computers.
- **HTML test reports:** New online mode (SETUP > Test reports) saves protocols with reduced space and faster saving process. However, these protocols can only be opened with Internet connection.
- **HTML test reports:** Suffix is added to report file names on USB-Stick to indicate the saving type: "-online" for the new online mode (see above), otherwise "-offline".
- **HTML test reports:** Comments entered during test steps will be added to the HTML test report.
- **System start:** Faster boot-up in case of huge database.
- **New feature User permissions:** Roles can be assigned to testing persons. Testing persons with the role "EIP" cannot change some settings. (Only available with feature "Z853S" or option "KD01").
- **SETUP:** Testing persons list is now sorted alphabetically.
- **SETUP:** Automatic test sequences can be configured in such a way that they are aborted immediately and without further inquiry in the event of a limit value violation. (Only available with feature "Z853S" or option "KD01").
- **SETUP:** Self test added for "Plug and protection class detection on test socket".
- **MEM:** When using "clone" for "devices" and "medical devices", the serial number will be deleted in the clone.
- **Only SECUTEST BASE/PRO:** Parameter box "SFC" is displayed on main screen if the rotary switch is positioned to the measurements "IT", "IPE", "IP".

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(12/8.19)

Improvements / Function Extensions

- First version with support of for **SECULIFE ST PRO test instrument**
- **MEM:** “Clone” feature for database objects (only with option Z853R or feature KB01)
- **USB flash drive:** Test report can be stored as HTML file (replaces BMP report printing feature).
- **Measurement:** “SFC” parameter has been added. Leakage current measurement with “direct method”: the following single faults can be selected:

I_{PE}	N interrupted
I_B	N interrupted PE interrupted
I_P	N interrupted PE interrupted Mains to application part
- **SETUP:** automatic test sequences are now configurable such that work can be continued even if limits are exceeded in order to get a complete error report for the DUT (only available with feature Z853S or option KD01).
- **SETUP:** “Culture” and “Features” appear now as sub-menus in the “System” menu.
- **SETUP:** Printer options for printer Z721S have been moved to the new “Test reports” menu item.
- **SETUP:** The “Barcode reader” menu item is now at the top level of the SETUP menu (previously: “External devices”).
- **Test reports:** A new customizable text box is now available for test reports.
- **Test reports:** option for more compact test reports by combining test steps within the report
- **Test reports:** option for minimizing the report view to “failed steps only” (for extensive test sequences)
- **Green rotary switch positions:** A freely definable comment can now be entered when saving measurement results to the database or sending them via the push-print function.
- **Test sequences:** additional for “SNR 462638” – selectable under SETUP – Autom. measur. – Meas. sequences
- **Integrated automatic test sequences:** Additional fuse check step at the end of every run.
- **R_{PE} measurement:** The range for the RPE “Offset” parameter has been extended from 2 to 5 Ohm (display range generally 27 Ohm instead of 30 Ohm now).
- **R_{PE} measurement:** The frequency parameter for RPE 200 mA AC is once again available.

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GB

Firmware Update to Version 3.0.0 (continued)

- **Orange rotary switch positions:** Pressing the “PRINT” key after saving test results now prints a single test report.
- **Visual Inspections:** Go/no-go responses to visual inspection prompts can now be changed directly by touch-click (only with touchscreen-option).
- **MEM:** MEM can now be controlled via an external keyboard (only available with option Z853S or feature KD01).
- **Language setup:** “Brazilian Portuguese” has been added to the culture menu.
- **RFID tags:** The problem which occurred while writing RFID tags with the Z751E has been solved.
- **MEM:** The problem which occurred when exporting certain databases has been solved.
- **XML export:** Filenames generated for XML export to USB memory devices will now include the test instrument’s serial number.
- **USB flash drive:** Filenames generated for database backups to USB memory devices will now include the test instrument’s serial number.
- **VDE 0701-0702/NEN 3140/ÖVE E 8701/SNR 462638:** new “IT mode (active)” sequence parameter for performing touch current measurement using the differential measurement method on DUTs with non-interruptible earth connection.
- **Orange rotary switch positions:** A “Skip” symbol is now displayed instead of the green triangle for inspection test steps with the “manual skipping possible” parameter (usually the on/off check for default test sequences).
- **MEM:** The MEM user interface can now (again) be exited via the START/STOP button.
- **Leakage current measurements:** secondary measured value “ULN” replaced with “ULPE”
- **Test reports:** Company logo can now be uploaded in full color and in various image formats (PNG, JPG, BMP, GIF).
- **Green rotary switch positions:** Measuring voltage can now be adjusted for RINS while conducting a measurement.
- **MEM:** The automatic DUT selection error after creation of a new DUT (via ID softkey or barcode/RFID scanner) has been fixed.

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GB

Firmware Update to Version 2.1.1 (11/10.18)

Improvements / Functional extensions

- **Firmware Update:** The data format for data export and import has been completely changed. Please check before updating the firmware of your test instrument as to whether your PC software is compatible with the current firmware version.
- **SETUP:** The file extension for data export to a USB flash-drive has been changed to „secu“. Direct import of data in the previous file format (file ending „.etc“) is no longer possible. Please proceed by importing the data into the IZYTRONIQ software as a first step, then export them and import them into your test instrument afterwards.
- **MEM – Database view:** The database view has been split into two separate perspectives (with release Z853R or feature KB01 only): customer view or location view. You can switch between the two perspectives by pressing the MEM key repeatedly. Press the ESC key to quit the database view.
- **MEM – Database hierarchies:** All database objects of type „Room“ or „Level“ must be subordinated to a database object of type „Building“ (the data migration generates object „Building“ if necessary). All database objects of type „Device“ or „ME Device“ must be created with a customer assignment (the data migration generates customer objects if necessary).
- **MEM – Mandatory fields:** For database object „Customer“ the entries in the fields ID and Designation, which are marked in red, are now mandatory (the data migration generates content if necessary). For the following database objects the entry for designation is a mandatory field which is highlighted in red: „Property“, „Building“, „Level“ and „Room“.
- **MEM – Search:** The search for ID (e. g. via RFID or barcode reader) now distinguishes between upper and lower case.
- **RPE:** The measurement with EL1 adapter for 10 A/25 A test current has been improved.
- **Interfaces – USB/Bluetooth:** The response to connection loss has been improved.
- **Short-circuit tests:** The L–N short-circuit test has been improved (the improved method might possibly not be activated until a readjustment has been performed). The limit value for the LN–PE short-circuit test has now been raised to 2 kΩ.

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GB

Firmware Update to Version 2.1.1 (Continuation)

- **SETUP:** The PC program „Report Designer“ is no longer supported. Adjustments of the test report and import of a company logo can now be directly performed in the SETUP of the test instrument (these settings must be re-entered after a firmware update).
- **Test as to whether the DUT is set up in a well-insulated fashion:** The test voltage has been reduced.
- **Integrated Test Sequences:** The standard safety class has been changed from „SK I“ to „SK I + II“ in order to capture conductive/metallic components which are not connected with the protective conductor.
- **VDE 0701-0702/NEN 3140/ÖVE E 8701:**
The default setting for test step „RISO SK II“ has been changed to „Individual measurement“ and the duration to „manual“ (0 s). The default setting for test step „IB“ has been changed to „Individual measurement“ and the duration to „manual“ (0 s).
- **IEC 60974-4:**
The notes on connection for the SECULOAD adapter have been corrected.
The limit values have been set to status IEC 60974-4:2016 .
- The **initial window** has been redesigned.
- Restoring the „**Default Settings**“ results in the additional deletion of the list of inspectors, the contents of the database and the saved company logo (for thermal printer test report).
- The „**Keyboard layout**“ configuration may be lost if an update is performed from firmware 1.8.x or 2.0.0 to one version above. Please make sure that the desired keyboard layout is still shown after the firmware update.

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(10/1.18)

Improvements

- Modified data format for data export and import. Please make sure that your PC software is compatible before updating your SECUTEST.
- Current clamp measurement has been supplemented in rotary switch position „EXTRA“ (only in models with feature I01).
- MEM: Database field „Safety Class“ for test objects (Device/ME device) has been added.
- MEM: Database field „Comment“ for database objects (Customer, Property, Building, Level, Room) has been added (only for release Z853R and/or feature KB01).
- Cursor-controlled screen keyboard/touch screen keyboard (touch screen only available for models with feature EA01): convenient entry of numbers due to special number entry mode.
- AUTO/A1-A8: classification parameters can be changed via touch screen (only available for models with touch screen).
- AUTO/A1-A8: switching to and from displayed test object information is possible per touch screen (only available for models with touch screen).
- As from this version, the firmware supports models with feature G02 (25 A RPE measuring current).
- Print-out of test reports: Setting for measuring uncertainty at the time of testing is printed as well.
- Switching parameters are marked with a special symbol.
- Label printer Z721D/Z721E: Print-out of 2D code labels
- Problem occurring while disconnecting DUT during visual inspection has been solved.
- Improved automatic setting of type size for question text in visual inspections
- AUTO/A1-A8: New SETUP option „Skip steps“
- USB flash drives: data backup files are saved in the root directory as from this version.
- „DB-Comfort“ release (Z853S), which is liable to costs, can be imported.
- The memory for user-defined test sequences has been extended to 24 sequences (only for release Z853R or feature KB01).

Functional extension with DB-Comfort release (Z853S) which is liable to costs

- MEM: Database field „test interval“ for test objects (Device/ME device) has been added (only for release Z853R or Z853S and/or feature KA01 or KD01).
- MEM: Database object type „Test object“ has been renamed as „Device“.
- MEM: „Medical electric devices“ are supported as object type in the database as from this version (only for release Z853S or feature KD01).

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GB

Firmware Update to Version 2.0.0 (Continuation)

- MEM: Shifting of database objects in the tree (only for release Z853S or feature KD01).
- AUTO/A1-A8: Pressing and holding the box with test object information triggers the processing/shifting of test objects (only for release Z853S or feature KD01).
- MEM: Entry option for additional DUT properties after creation via ID softkey or barcode/RFID scanner (only for release Z853S or feature KD01).
- Push/print function: Transmission of measuring/test results per keystroke via USB (or Bluetooth, if available) (only for release Z853S or feature KD01).
- AutoStore: Save results automatically or send them per push/print if a DUT is connected (only for release Z853S or feature KD01).
- MEM: As from this version, the search function allows for searching in the „UDI“ field of ME devices as well (only for release Z853S or feature KD01).

firmware Update to Version 1.8.3 (9/12.17)

- Integration of printer Z721E (new barcode printer replaces Z721D)

Firmware Update to Version 1.8.2 (8/6.17)

Test Sequences / Sequences

- All integrated test sequences: The test steps RPE, RISO (SK II) and IB can now be changed to „Continuous measurement“ (Parameter: Measurement duration = 0).
- For measurement type passive the measurement of SELV voltage (PC III) has been canceled in all test sequences.
- In the SETUP, the setting „At end of sequence: Results list“ has been reactivated.

Calibration

- The calibration and recalibration date stored to the instrument can be set via the interface.
- The recalibration date stored to the instrument can also be adjusted manually in the SETUP.

Working with Accessories

- Measurement of IPE and IG: The clamp factors 10:1, 100:1 and 1000:1 have been added (compatibility with accessory leakage current clamp sensor SECUTEST CLIP (Z745H)).

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GB

- The testing of multiple socket outlets via the AT3-III-E adapter has been optimized.

Database, Report Generation

- Database backups created with previous firmware versions can now be restored.
- The display error in the MEM detail view has been corrected.
- The description of RFID tags with Z751E has been improved.
- Compatibility with USB storage devices has been enhanced.
- The printing of test reports on USB flash drive has been improved.
- Corrections have been applied to the English test report.

Further Improvements

- The test socket is no longer switched off
 - when entering the U(R) limit value,
 - when entering the power rating of the heating element during IPE measurement
- The numeric OnScreen keyboard has been improved for the French/Czech language version
- Resetting to „Default settings“ now also deletes the company logo.
- The on-test has been optimized.

- **After this update you have to update too: SequenceDesigner to version 1.5**

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© Gossen Metrawatt GmbH
All trademarks, registered trademarks, logos, product names, and company names are the property of their respective owners.
Gossen Metrawatt GmbH